



ergo vac compact

Supplement to the operating instructions for ergo vac compact

Art No. 360901

Page 33

8.1 Possible errors

Electronic errors

customer information

i

Please read the following supplement to the operating instructions before using the device.

In the worst case, the system can be blocked in the battery saving mode function. This is manifested in the fact that after the battery saver mode has been triggered, the device cannot be reset to the active state even after charging with the AC adapter.

This state can be reset by the following steps:

1. Connect the system to the original power adapter and make sure the system has been charged for at least 30 minutes .
2. Check the health of the system. Press the power button. Battery saver mode triggered -> left LED flashes. System cannot be started despite power connection.



3. Press the on/off button three times.



4. Check the functions of the suction system.

i

If the system still does not start after the reset, there is likely an intake system fault that needs to be addressed. In this case, take the system out of operation and send it to the mbnet repair service for inspection.

ergo vac compact bears the mark certifying that it complies with the applicable essential safety and performance requirements set out in Annex I of Regulation (EU) 2017/745 on medical devices (MDR). The requirements relate to patients, users, and third parties who come into contact with this device during its intended use.

Stand: 10/2022

mbnet Engineering GmbH
Kirschauer Strasse 37a
OT Callenberg
D-02681 Schirgiswalde-Kirschau

Telefon +49 (0)3592 34 83 0
E-Mail info@mbnet.de
Internet www.mbnet.de